

VAKU-8 +

DIRECTIONS FOR USE

SAFETY BLOOD COLLECTION SYSTEM & INFUSION SET


- 1** OPEN THE PACKAGE AND REMOVE THE BLOOD COLLECTION SET AND GENTLY PULL THE TUBING FREE OF THE KEYHOLE NOTCH IN THE SAFETY GUARD



- 2** PREPARE THE VEINIPUNCTURE SITE, REMOVE THE NEEDLE COVER AND INSERT THE NEEDLE INTO THE PATIENT.



- 3** BEFORE REMOVING THE NEEDLE FROM THE PATIENT, SLIDE THE SAFETY SHIELD SO IT IS TOUCHING THE BACK OF THE WINGS

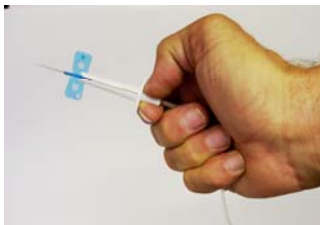

- 4** GENTLY PRESS GAUZE OVER THE WOUND BEFORE REMOVING THE NEEDLE



- 5** WITH THE OPPOSITE HAND PLACE THE TUBING BETWEEN YOUR THREE BACK FINGERS AND YOUR PALM AND HOLD THE "GRASPING AREA" OF THE SAFETY SHIELD BETWEEN YOUR THUMB AND FOREFINGER

- 6** BEGIN WITHDRAWING THE NEEDLE FROM THE PATIENT AND SIMULTANEOUSLY SLIDE SHIELD FORWARD OVER THE WINGS BY EXTENDING YOUR THUMB AND FOREFINGER WHILE HOLDING THE TUBE STATIONARY BETWEEN YOUR BACK THREE FINGERS AND PALM

- 7** ***WHEN IT 'CLICKS' ITS SAFE***

*Remember to discard the used needle
in an approved sharps container!*



Bringing Value, Reliability & Innovation to Healthcare...®