

TheSharpChoice®

NMSDC Certified Diversity Supplier

GLASSVAN® PLUS


Cartridge Blade Management System




The GLASSVAN® Plus Cartridge Blade Management System provides the weight and feel of a traditional scalpel handle with the added features of a safety scalpel.

- Unique safety cartridge device that attaches directly to a standard scalpel handle resulting in a secure fit
- Protective sliding sheath ensures safe handling and disposal
- Simple detachment of blade after use
- Designed to fit most handles that conform to the ISO 7740 standard
- Meets OSHA Bloodborne Pathogens and Needlestick Prevention guidelines
- Sterile and supplied ready for use with high durability Swiss stainless steel blade


BRAND (Company)	Blade Attaches Directly to Standard Handle	Protective Sliding Sheath	One-handed Activation	Ambidextrous Use
GLASSVAN® Plus Cartridge Blade Management System (MYCO Medical)	✓	✓	✓	✓
Bard-Parker™ Safety Blade (Aspen)		✓	✓	✓
Cabo Safety Scalpel Cartridge (Southmedic)		✓	✓	✓

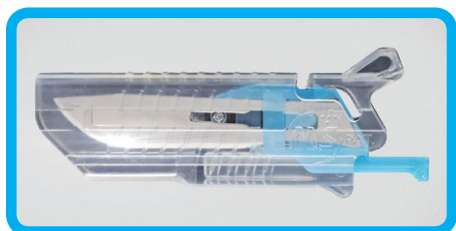

Unique safety cartridge

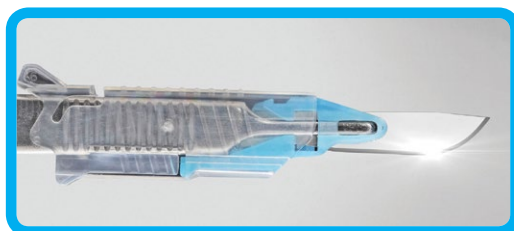

Surgical blade attaches directly to scalpel handle

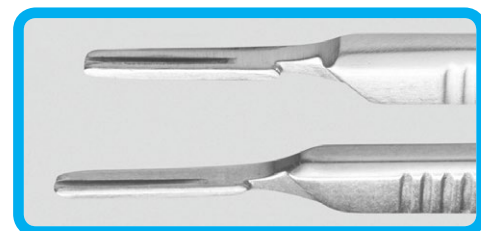

Fits most standard handles

Instructions for Use

SUPPLIED READY FOR USE, WITH STAINLESS STEEL BLADE


ATTACHING THE GLASSVAN® PLUS CARTRIDGE-BLADE MANAGEMENT SYSTEM.

1. Remove from packaging. (*Sterile only if package is NOT opened, damaged or broken. Intended for single use only. Do NOT re-sterilize.*)
2. Insert scalpel handle (B) into the back end of the Blade Management System (A). Ensure the handle is turned in the correct direction so as to engage into the blade as with typical blade loading.
3. Push handle (B) forward – it will be guided towards the blade (C) and will engage the blade. NOTE – the surgical blade will attach to the handle exactly as a blade would attach directly to a standard scalpel handle.
4. Push the handle fully forward until a loud “click” is heard. Check VISUALLY to ensure the blade is attached. If the blade is properly attached, the handle cannot be pulled out of the cartridge.




EXPOSING THE BLADE

5. Place thumb on sheath clip (D). Pull back with slight downward force to expose blade for use. Pull sheath backwards towards rear of the scalpel handle as far as it will go. Apply slight force to lock (F) to prevent movement of sheath during a procedure. (This is only required if grip is solely on sheath and a stabbing force is applied).

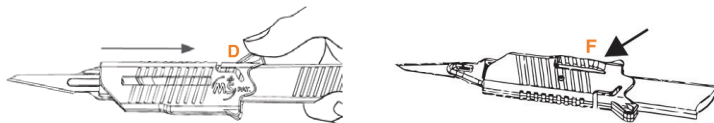


CLOSING THE SHEATH AND COVERING THE BLADE

6. Place thumb behind sheath clip (D). Push forward until a loud click is heard and felt. Visually confirm the sheath has fully covered the surgical blade.
7. Repeat process as required during use and passing.

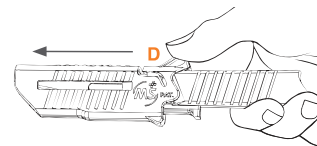

DETACHING THE GLASSVAN® PLUS CARTRIDGE-BLADE MANAGEMENT SYSTEM

8. **Two-handed method (ambidextrous)** – Hold scalpel handle in either hand with blade pointing away from your body and with sharp edge facing right. With right hand thumb on blade-removing lever (E) and left hand thumb on sheath clip (D), push down and forward simultaneously. Blade Management System will safely eject from handle with blade safely housed within the sheath.
9. **One-handed method (right hand only)** – Hold scalpel in right hand with blade pointing away from your body and with sharp edge facing right. With thumb on sheath clip (D) and forefinger on blade-removing lever (E), push down and forward simultaneously. (Note – the lever pushes back of the blade out and away from the handle with used blade safely housed within the sheath).

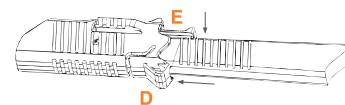

SHARPS DISPOSAL

10. This may be done directly from the scalpel handle into a sharps safety container or per your individual facility protocol.

MYCO CODE	BLADE SIZE	UNIT OF MEASURE
02494	#10	50 cartridges/bx; 20 bx/cs
02495	#11	50 cartridges/bx; 20 bx/cs
02496	#15	50 cartridges/bx; 20 bx/cs

As a leading NMSDC Diversity Certified Supplier, MYCO Medical is committed to “Bringing Reliability, Innovation and Value” to the Healthcare industry with an emphasis on “physician preference” surgical devices and disposables.

To order, learn more or receive complimentary evaluation samples:

Please call MYCO Medical at (800) 454-6926 or (919) 460-2535 or email us at info@mycomedical.com



2015 Production Drive
Apex, NC 27539 USA

info@mycomedical.com
www.mycomedical.com

See the complete line of MYCO innovative disposable and reusable medical devices at www.mycomedical.com

Some medical devices are not licensed for sale in Canada.

© 2017 MYCO Medical, Inc. All rights reserved.
LIT-00106 Rev 4 Eif 05.17.2017