

# QLICKSMART® BladeFLASK

## Scalpel Blade Removal System

The QLICKSMART® BladeFLASK is the world's first single-handed scalpel blade remover.



- Holds 100 blades and is puncture proof
- OSHA compliant with single-handed activation when used with reusable QLICKSMART® BladeFLASK Mounting Bracket (*sold separately*)
- Mount on a wall or desk with the reusable QLICKSMART® BladeFLASK Mounting Bracket (*sold separately — adhesive backing and screw mounts included*)

- Audible click ensures blade removal
- Built-in counter with automatic shutoff when full
- Compatible with most brands of general surgery blades and flat handles



MYCO CODE	DESCRIPTION	UNIT OF MEASURE
QFRGEN-01	BladeFLASK – Non-Sterile	12 ea/case
QBRGEN	BladeFLASK Mounting Bracket – Non-Sterile (Reusable, includes adhesive backing and screw mounts)	24 ea/case

## How to Use:

### PARTS OF YOUR BLADEFLASK:

**INDICATOR:** SHOWS WHICH WAY SHARP EDGE MUST FACE.

THIS PART OF THE SCALPEL HANDLE IS CALLED THE TANG.

COUNTER

LANDING PAD

MOUNTING BRACKET

### MOUNT TO WALL OR TROLLEY:

- If you haven't mounted your BladeFLASK before, prepare mounting bracket.
  - BRACKET HAS DOUBLE SIDED TAPE ON BOTTOM;
  - OR USE SCREWS.

\* See fitting template and instructions inside the bracket packaging.
- BladeFLASK can be mounted on a wall or on a trolley or bench.

SLIDE DOWN INTO BRACKET

SLIDE BACK IN TO BRACKET

**NOTE:** BladeFLASK is mounted on Mounting Bracket during use.

### REMOVE BLADE SINGLE HANDED ACTION:

- Place blade on landing pad with the sharp edge facing to the right (or 'tang' up).
- Push scalpel handle into BladeFLASK until you hear a click.

**CLICK!**
- Remove handle.

**NOTE:** BladeFLASK is disposed of as medical waste after 100 blades have been inserted.

As a leading NMSDC Diversity Certified Supplier, MYCO Medical is committed to “Bringing Reliability, Innovation and Value” to the Healthcare industry with an emphasis on “physician preference” surgical devices and disposables.

## To order, learn more or receive complimentary evaluation samples:

Please call MYCO Medical at (800) 454-6926 or (919) 460-2535 or email us at [info@mycomedical.com](mailto:info@mycomedical.com)

See the complete line of MYCO innovative disposable and reusable medical devices at [www.mycomedical.com](http://www.mycomedical.com)



**MYCO**  
MEDICAL  
TheSharpChoice®

2015 Production Drive  
Apex, NC 27539 USA

[info@mycomedical.com](mailto:info@mycomedical.com)  
[www.mycomedical.com](http://www.mycomedical.com)

Some medical devices are not licensed for sale in Canada.

© 2017 MYCO Medical, Inc. All rights reserved.

LIT-00217 Rev 2 Etf 10.24.2017