



MYCO
MEDICAL

TheSharpChoice®

NMSDC Certified Diversity Supplier


RELI® SAFETY BLOOD COLLECTION SET

Safety Sheath Activation Instructions for Use

Reli® Safety Blood Collection Sets feature single-handed safety shield activation for convenience and ease of use. The following steps should be taken to properly use the Reli® Safety Blood Collection Sets:

STEP 1:

Use wings to perform venipuncture. Flashback will occur where tubing begins.

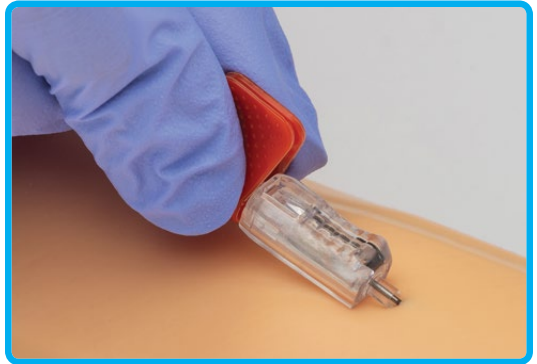

STEP 2:

Place index finger on curved design, press **GENTLY** while using thumb and middle finger to pull back on wings until 1st “click” is heard.



STEP 3:

Continue to pull back until 2nd “click” is heard, indicating the needle is locked, protected and ready for disposal.



Reli[®] Safety Blood Collection Sets from MYCO Medical are available in 21G, 23G, and 25G needle gauges, in a variety of tube lengths, with or without tube holder.

To order, learn more or receive free evaluation samples:

Please call MYCO Medical at (800) 454-6926 or email us at customerservice@mycomedical.com

Pictured above: Reli[®] Safety Blood Collection Set (GSBCS23G-7T)



Pictured right: GSBCS25G-12

Pictured left: GSBCS21G-3T



2015 Production Drive
Apex, NC 27539 USA
info@mycomedical.com
www.mycomedical.com

Some medical devices are not licensed for sale in Canada.

© 2019 MYCO Medical, Inc. All rights reserved.
LIT-00654 Rev 0 Eff 12.16.2019